



# HELP LEAVE APPLIANCES CLEAN AND FRESH\*

WITH

**POLIDENT**

**PRO GUARD**  
& RETAINER



Daily clean



For patients with removable oral appliances

\*when used as directed

# ► DON'T JEOPARDIZE YOUR SMILE!



Clean your removable oral appliance daily to:

- Help reduce odor-causing bacteria
- Help leave appliances clean and fresh\*



\*When used as directed.

## CLINICALLY PROVEN TO HELP KEEP YOUR RETAINER, ALIGNER OR MOUTH GUARD CLEAN

**POLIDENT**

# SUITABLE FOR CLEANING REMOVABLE ORAL APPLIANCES\*\*

- ◆ Kills 99.99% of odor-causing bacteria\*
- ◆ Proven to help keep retainer, aligner or mouth guard clean
- ◆ Lifts stains, removes discoloration
- ◆ Cleans effectively while being gentle on a range of dental appliances†

Aligner



PVC vacuum formed  
mouth guard



TESTED ON  
**20**  
DIFFERENT  
MATERIALS\*\*



Retainer



Polycarbonate  
mouth guard



Wired retainer

\*In laboratory tests  
\*\*Based on *In Vitro* testing.

†Ask your dental professional  
if Polident is suitable for your  
dental appliance.

††This does not represent the  
full compliment of materials  
on the market.



## ► **ADVICE FOR A DAILY CLEAN**



Drop one Polident Pro Guard & Retainer cleanser tablet into enough very warm (not hot) water to cover your oral appliance.



Soak for 3 – 5 minutes. For best results, gently brush your oral appliance with cleanser solution using a soft brush. Discard remaining solution immediately after use.



Rinse appliance thoroughly with running water.



**USE DAILY TO HELP STOP CUMULATIVE BUILD  
UP OF ODOR-CAUSING BACTERIA TO HELP  
KEEP YOUR DENTAL APPLIANCE CLEAN\***

Please read the full label on back of pack for full instructions.

Get more tips and advice at  
**[www.polident.com](http://www.polident.com)**

\*In laboratory tests.

Trade marks are owned by or licensed to the GSK group of companies. PM-US-POLD-20-00113